



**Seamless quality**  
**For lasting comfort**  
**TENA *Fix* Premium**



# More comfortable

# More washable

# More secure



## Seamless comfort

TENA *Fix Premium* has no side seams. With no pressure points, wearers are assured comfort which helps avoid skin chafing and irritation.

## Stable and secure

Using TENA *Fix Premium*, you experience a stable and secure fixation of TENA *Comfort* products – for comfort, performance and peace of mind.

## Just like underwear

TENA *Fix Premium* is made from a close-mesh, 96% polyester, 4% elasthan material – that feels as comfortable and familiar as normal underwear.



## Self-assisted



1 Pull fixation pants to mid-thigh then turn waistband down to knees. Fold TENA *Comfort* lengthways and form a bowl.



2 Place TENA *Comfort* in position from front to back.



3 Pull up fixation pants, adjusting leg elastics to groin area.



## Identify each wearer

To help you manage the products there is a space on the waistband to write the name of the wearer or room number using a waterproof marker.

## Resistant to washing/drying

The design and material ensure you can now wash a pair of fixation pants up to 50 times at 60°C. You can also tumble dry them.

## Easy to select

Colour-coded according to size for easy and accurate selection.


## Dependent



1 Pull fixation pants to mid-thigh then turn waistband down to knees. Fold TENA Comfort lengthways and form a bowl.





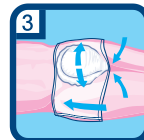



2 Place the TENA Comfort from front to back and ensure front and back are smoothed out.



3 Pull up fixation pants, adjusting leg elastics to groin area.

# The new TENA *Fix* Premium range

	Size	Art. no.	Pack count
	XS	754034	8x25
	XS	754033	20x5
	S	754028	8x25
	S	754023	20x5
	M	754029	8x25
	M	754024	20x5
	L	754030	8x25
	L	754025	20x5
	XL	754031	8x25
	XL	754026	20x5
	XXL	754032	8x25
	XXL	754027	20x5

For more information go to [www.TENA.co.uk](http://www.TENA.co.uk).



SCA HYGIENE PRODUCTS UK LIMITED,  
 Personal Care, Southfields Road  
 Dunstable, Bedfordshire LU6 3EJ  
 Tel: 01582 677400  
[www.TENA.co.uk](http://www.TENA.co.uk)

*c/o Life* Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.