



iD EXPERT SLIP FITTING GUIDE



YOUR TRUSTED PARTNER
IN CONTINENCE CARE



iD EXPERT SLIP

iD Expert Slip are All-in-One briefs which offer an effective solution for managing moderate to heavy incontinence. With a high level of absorption, the All-in-Ones provide the user with anti-leakage security and optimal comfort. They are particularly recommended for people with limited mobility or who are confined to bed.



STEP 1

Start by folding the iD Expert Slip in half lengthways and gently channel it by running your fingers underneath the backsheet. This activates the anti-leak cuffs.

STEP 2

If the client is immobile, gently roll the patient onto their side with one leg slightly raised to prevent them from falling. With safety in mind, position the pad at the patient's coccyx and ensure that the end with the fixation tapes is placed at the back.



If the patient is mobile, pass the iD Expert Slip between the legs from front to back and fan out the ends, ensuring the top rear end is positioned at the coccyx. Make sure that the end with the fixation tape is placed at the back.

Ensure the pad is centred correctly using the wetness indicators as a guide.



STEP 3

If the patient is immobile, roll them onto their back and gently pull the product up through the legs, ensuring the anti-leak cuffs remain activated and then fan out the product. If necessary, roll the patient over onto their side again to ensure the back of the product and the fixation tabs are easily accessible and in the correct position.


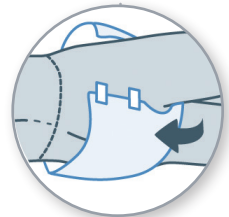
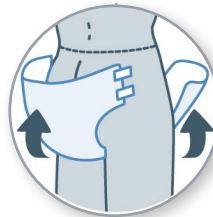


STEP 4

Fix the bottom tapes in an upward position assuring a secure fit about the legs and groin.

Fix the resealable top tapes in a downward or horizontal position to ensure a secure fit about the waist.

Finally, check for comfort and fit.

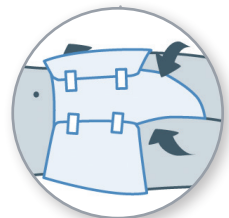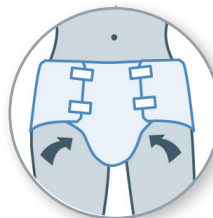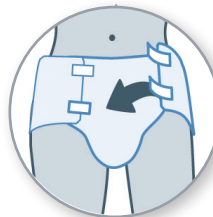



WETNESS INDICATOR

Each pad possesses a wetness indicator which you can identify by looking for two coloured lines (one yellow and the other blue). These are a guide as to when you need to change the product and can be checked without the need to remove the pad.

As you pass urine into the pad the yellow line will gradually turn blue and the blue line will smudge and fade. Once the majority of the lines have changed it is time for you to change and dispose of your pad.

IMPORTANT ADVICE

The use of talcum powder is not recommended, oil based barrier creams may reduce the performance of the pads. If the barrier creams must be used they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner. They cannot be flushed down the toilet.

iD EXPERT SLIP

AVAILABLE WAIST SIZES:

EXTRA SMALL	40-70CM
SMALL	50-90CM
MEDIUM	80-125CM
LARGE	115-155CM
EXTRA LARGE	120-170CM

