




## SLIP ACTIVE




Attends Slip Active are a range of all-in-one products designed to manage heavy urinary and / or faecal incontinence.


### Fitting Guide

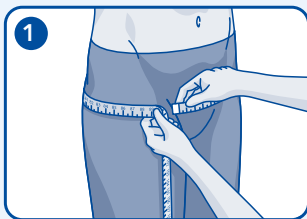


1 Measure the hip and waist and choose the larger of these two to select the correct product size.

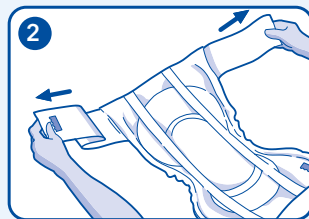

2 Open the product fully and position the end with the wings and fixation tapes to the back. Make sure the wings are also fully opened and that the back sheet is on the outside.

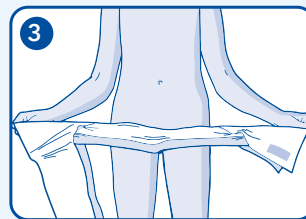


3 Gently fold and cup the product lengthways to ensure that the leakage barriers stand up.

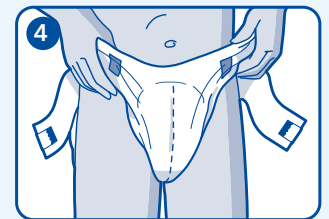


4 Position the product between the legs so that it fits snugly to the wearer and that it fits comfortably in the groin area. Use the wetness indicator to help to position and centre the product.



5 Gently pull the wing around to the front of the product. Use the blue positioning tape to pre-apply the product on both sides.

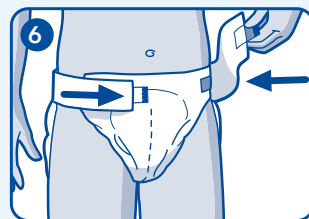

6 Fasten the tapes firmly onto the product. When you remove the product lift the white tape off the blue tape ensuring that the blue tape remains on the waterproof backsheet.

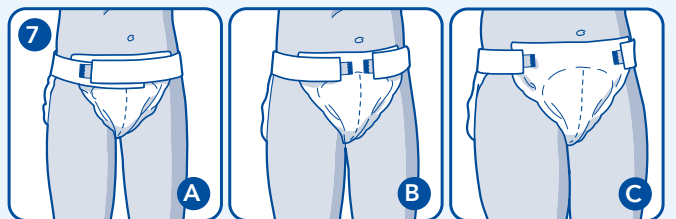

7 Depending on the anatomy of the user the wings may:  
**A.** Overlap  
**B.** Be close to each other  
**C.** Be close to the edge of the Slip

# SLIP ACTIVE

## Hints and Tips

### Selection

- Consider selecting Attends shaped pad/pants before using Slips - they offer much greater discretion.
- Choose the correct product size according to the hip and waist size and take whichever is the largest.
- Choose the absorbency based on the volume of urine passed. Start by choosing a lower absorbency number before moving to a higher absorbency.
- Slips can offer effective management for people who may pull out shaped pads.

### Fitting

- Ensure you keep the ruffles on the flexible leg gathers on the outside of the product - this keeps the waterproof backsheet away from the skin.
- Keep the adhesive tapes at the back when fitting the product - you then know it is the correct way round.
- If the product is fitted when the user is laying down follow the same sequence as for fitting the product in a standing position. To make sure the product remains in place, place a hand on the back of the product when turning the client.
- Use the wetness indicator to centre the product.

### Changing

- Check the wetness indicator - if the wetness indicator has disappeared over half the length of the product (60%) or if it contains faeces the product requires changing. If not the product can be reapplied.
- Clean and dry products can be resealed and reused after toileting.
- When you unfasten the tapes the blue tape should remain in place on the product.
- Clean skin as per your local policy. Clean from front to back.
- Only use prescribed creams and apply as per the manufacturer's instructions.

### Leakage

- Check all aspects of fitting and assessment.
- Check that you have the correct size.

### Disposal

- Roll up the product and dispose of as per your local authority or NHS waste policy.
- Do not flush the product down the toilet.



**Skin tolerability** is dermatologically approved by SGS proderm Institute for Applied Dermatological Research