




ADJUSTABLE




Attends Adjustable are all-in-one products with a flexible fixation system which allows the product to be fitted in a number of ways. Designed to manage heavy urinary and/or faecal incontinence.

Fitting Guide


The following three ways of fitting the product can also be used if the user is sitting or lying down.

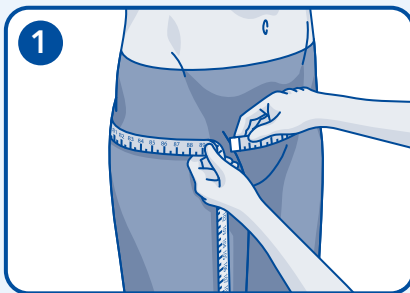


1 Measure the hip and waist and choose the larger of these two to select the correct product size.

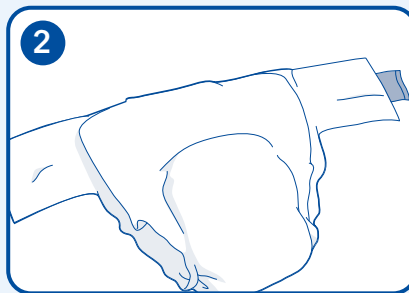


2 Open the product fully and position the end with the wings and fixation tapes to the back. Make sure the wings are also fully opened and that the back sheet is on the outside.

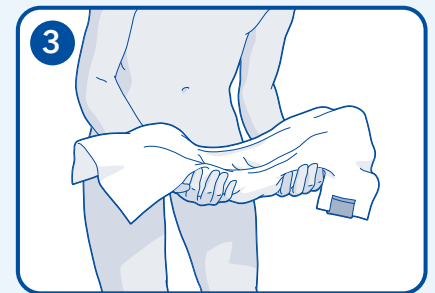


3 Gently fold and cup the product lengthways to ensure that the leakage barriers stand up.

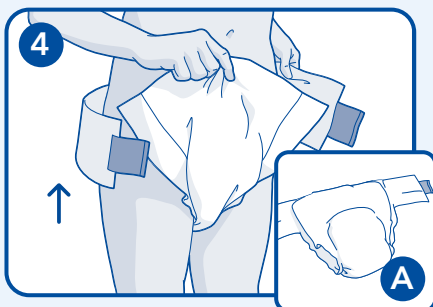

Option A - Leave the fixation tapes open on both sides. Position the product between the legs so that it fits snugly to the wearer and comfortably in the groin area. Use the wetness indicator to help to position and centre the product.

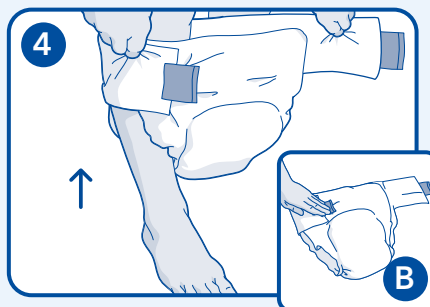


Option B - Fasten the fixation tape on one side before fitting the product. Step into the product with one leg and pull the product up so that it fits comfortably in the groin area. Use the wetness indicator to help to position and centre the product.

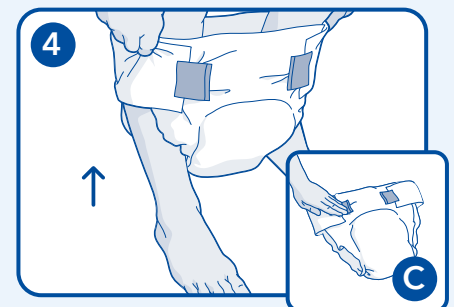


Option C - Fasten the fixation tapes loosely on both sides before fitting the product. Step into the product as if it were a pant and pull it up so that it fits comfortably in the groin area. Use the wetness indicator to help to position and centre the product.

Attends®


Incontinence protection for you

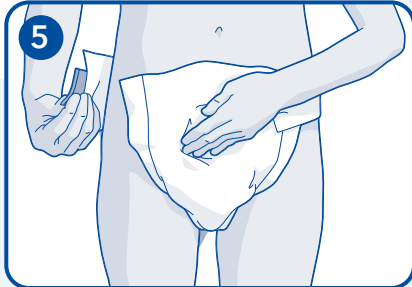

Gently pull the wing around to the front of the product. Firmly press the fixation tapes anywhere on the front panel to secure the product in place.

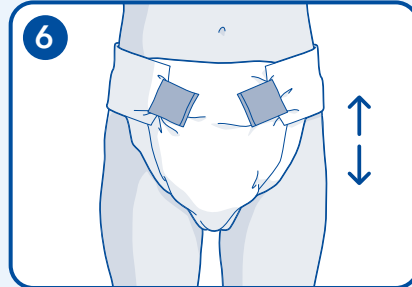

The fixation tapes can be readjusted as required to give an individual fit for the user and can be opened, readjusted and resealed.

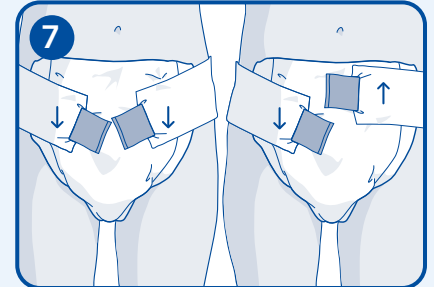

The fixation tapes can be configured in different ways to get the best fit for the user.

Hints and Tips

Selection

- Choose the correct product size according to the hip and waist size and take whichever is the largest.

Fitting

- Gently fold and cup the product lengthways to ensure that the leakage barriers stand up.
- Ensure you keep the ruffles on the flexible leg gathers on the outside of the product.
- Keep the fixation tapes to the back when fitting the product - you then know it is the right way round.
- Use the wetness indicator to centre the product on the body.
- The fixation tapes can be placed anywhere on the front panel. Press the tapes firmly on the product to secure it in place.
- The fixation tapes can be readjusted as required to give an individual fit for the user and can be opened, readjusted and resealed.

Changing

- Check the wetness indicator - if the wetness indicator has disappeared over half the length of the product (60%) or if it contains faeces the product requires changing. If not the product can be reapplied.
- Clean and dry products can be resealed and reused after toileting.
- Remove soiled products from the rear where possible.
- Clean skin as per your local policy. Clean from front to back.
- Only use prescribed creams and apply as per the manufacturer's instructions.

Leakage

- Check all aspects of fitting and assessment.
- Check that you have the correct size.

Disposal

- Roll up the product and dispose of as per your local authority or NHS waste policy.
- Do not flush the product down the toilet.



Skin tolerability is dermatologically approved by SGS proderm Institute for Applied Dermatological Research