



## INSTRUCTIONS FOR USE




**1** Adjust the bed to ensure patient is comfortable for toileting. VernaFem is more effective when the patient is in an upright position.

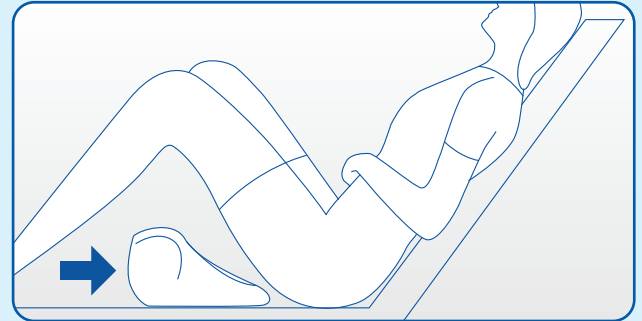


**2** Position the VernaFem ensuring a comfortable fit against the patient's body. If the patient is able, they may position the product themselves, with no patient lifting required.  
 Ensure the patient does not sit on the product.

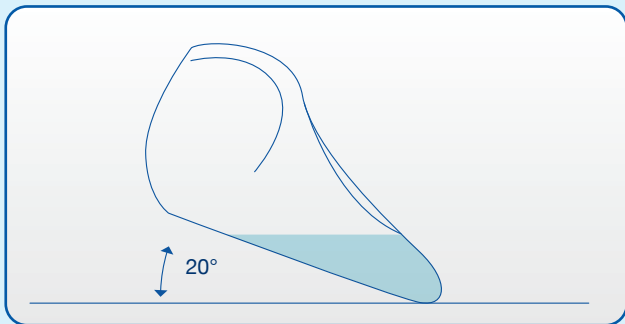


**3** The usable capacity is 900ml, with a usable capacity of 350ml when held at 20° angle. Vernagel may be used to reduce the risk of spillages.

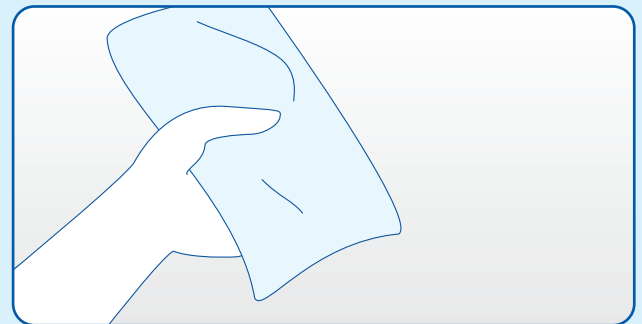

**4** After toileting, use tissue paper or a soft wipe (ideally maceratable) to wipe away any excess urine from the patient.

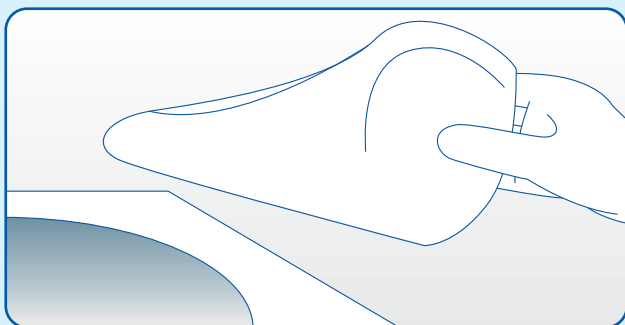

**5** Carefully remove the VernaFem using the handle and dispose of in a pulp disposal unit (such as a Vortex/Vortex+).

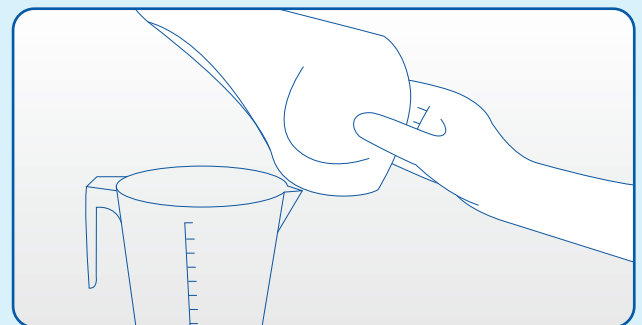

**6** If required, the handle of the VernaFem can be used to pour the urine into a measuring jug prior to disposal.

### Have you considered?

#### Vernagel Superabsorbent Powder

Order Code: 450SC100 NPC Code: HFL8662


Minimises risk of spillage / Eliminates residual odours




## INSTRUCTIONS FOR USE




**1** Adjust the bed to ensure patient is comfortable for toileting. VernaFem is more effective when the patient is in an upright position.

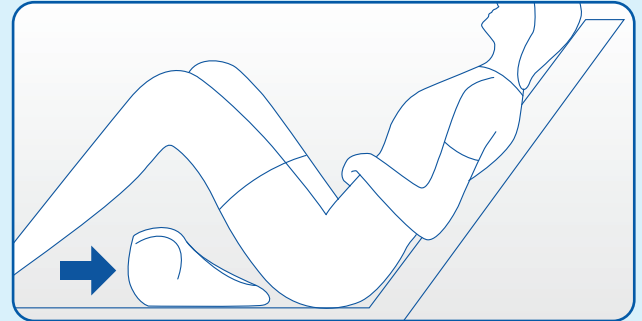


**2** Position the VernaFem ensuring a comfortable fit against the patient's body. If the patient is able, they may position the product themselves, with no patient lifting required.  
**Ensure the patient does not sit on the product.**

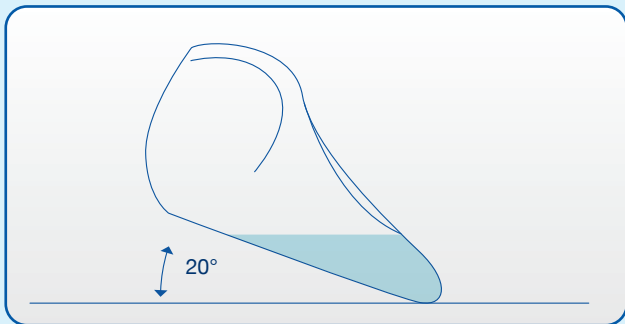


**3** The usable capacity is 900ml, with a usable capacity of 350ml when held at 20° angle. VernaGel may be used to reduce the risk of spillages.

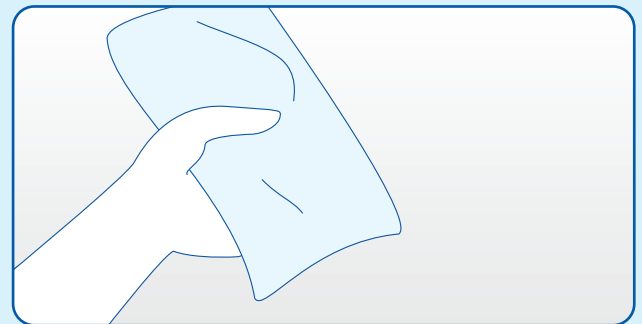

**4** After toileting, use tissue paper or a soft wipe (ideally maceratable) to wipe away any excess urine from the patient.

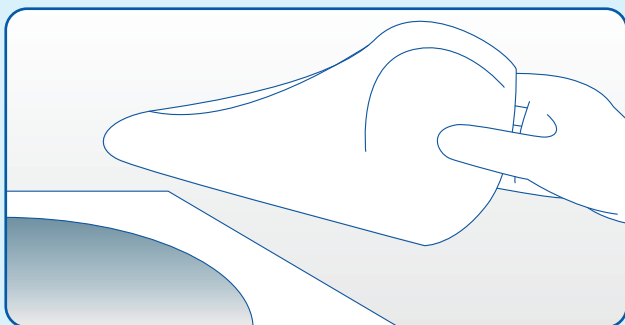

**5** Carefully remove the VernaFem using the handle and dispose of in a pulp disposal unit (such as a Vortex/Vortex+).

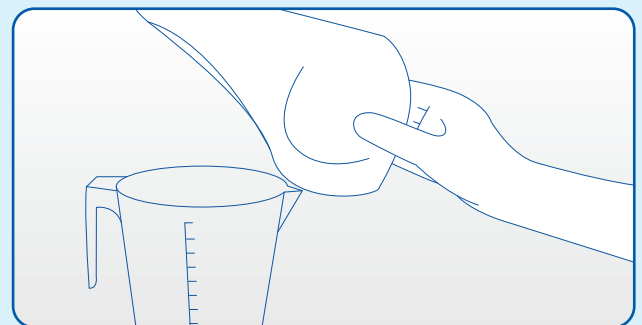

**6** If required, the handle of the VernaFem can be used to pour the urine into a measuring jug prior to disposal.

## Have you considered?

### Vernagel Superabsorbent Powder

Order Code: 450SC100 NPC Code: HFL8662

Minimises risk of spillage / Eliminates residual odours

